## How does Papiopad® work against warts?

A wart is caused by a viral infection of the skin. Fever helps the body remove viral infections. By using the Papiopad® heat compresses, the skin is heated to just the right temperature to cause localized "fever".

The body then removes the virus so that the skin recovers in three months and the wart disappears.

Papiopad® heat compresses contain sodium ethanoate hydrate. Direct contact with skin is not dangerous; simply rinse the skin with water.

## Question and Answer

**Question: Why can it take several months before the wart is gone?**

**Answer:** The skin tissue of the wart is replaced by healthy skin in two to three months.

**Question: Are warts contagious?**

**Answer:** Yes. Warts are caused by a viral infection of the skin. The skin shed from the wart will carry the virus and therefore be infectious.

**Question: I have many warts. Do I have to treat every wart?**


**Answer:** You don't always have to treat every wart. Often if you treat the largest or most irritating warts, the rest of the warts will also disappear. Any remaining warts can be treated again with Papiopad.

**Question: If a Papiopad® heat compress begins to leak, is that dangerous?**

**Answer:** No. The Papiopad® heat compresses contain a non-toxic salt solution. Rinse the salt away with water. The broken Papiopad® heat compress can be disposed of as household waste.

Registered by:

Produvation BV, Delft, The Netherlands  
2014-Class I-NL-CA002-2014-31371  
European Directive 93/42/EEG

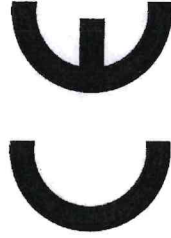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**hittecompressen  
tegen wratten**

### Read this manual before use

Papiopad® Foot has been developed for the treatment of plantar warts. The use of both Papiopad® heat-compresses once every two days for a period of 3 weeks is recommended. The heat compresses are reusable up to 50 times.

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### contents of your package

The Papiopad® Foot package consists of two Papiopad® heat compresses, two compression sleeves, a bandage and this manual. Papiopad® Foot has Class I medical certification according CE marking directives of the EEA.

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[info@meditech-europe.nl](mailto:info@meditech-europe.nl)

## Use

Remove the cellophane around the compress.

A metal disc is visible in the clear liquid in the compress. By clicking the metal disc back and forth (feel a click), the liquid crystallizes and the heat compress immediately becomes hot.

If you cannot press the metal disc, try pressing it from the other side of the disc. If you have "clicked" the disc but the compress does not heat up, press more often until the compress becomes hot. When using a new compress for the first time, you will sometimes have to click a few times before it gets warm. If a compress is reused, a single click is almost always sufficient.

After activating, knead the compress briefly until the compress turns milky and smooth and then put the compress in the bandage.

Afterwards press the activated heat compress firmly on the area of the wart for 15 minutes.

After these 15 minutes, replace the heat compress immediately with the second activated compress.

After another 15 minutes the treatment is ready for that day. In total you heat the infected spot 30 minutes at a time. For warts, use of both Papiopad® heat compresses once every two days for a minimum period of 3 weeks is recommended.

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## Attention!

The use of the Papiopad® heat compresses on the skin should not be painful. If heating is painful, slide the supplied compression-sleeves over the Papiopad® heat compress. This will lower the temperature of the compress on the skin. If the pain persists, do not continue. **Users with poor circulation and disrupted pain sensation, such as people with diabetes and neuropathy, are advised to use Papiopad® only in consultation with their physician.**

## Reusing Papiopad®

After cooling, the used heat compresses are hardened. You can ready them for reuse by boiling the compresses in water.

Proceed as follows:

- 1- Put the hardened compresses in a pan with enough water to submerge the compresses.
- 2- Bring the water to a boil and continue to boil for 5 minutes until all crystals inside the compresses have dissolved. The compresses need to be completely clear again.
- 3- Allow the compresses to cool to room temperature. The compresses are then ready for reuse. Keep the compresses in a clean place.

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